A-dec ICX Renew is a specially formulated dental unit waterline shock treatment. It is intended to lower bacterial* contamination in effluent and remove build-up of non-pathogenic microbial contamination from the waterlines. Easy to use and no diluting required, ICX Renew cleans dental unit waterlines and removes the buildup of odor-causing, foul-tasting bacteria. Always use ICX Renew Solution 1 and 2 together. Do not add water. This product is to be used in conjunction with regular testing of dental unit water.

**Non-public health bacteria.*

KEEP OUT OF REACH OF CHILDREN

CAUTION

PRECAUTIONARY STATEMENTS. Solution 1 and 2:

Hazards to Humans and Domestic Animals

CAUTION. Causes moderate eye irritation. Harmful if absorbed through skin. Avoid contact with eyes, skin, or clothing. Wear protective eyewear (goggles, face shield, or safety glasses) and rubber gloves when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

FIRST AID. Solutions 1 and 2:

Have product container or label with you when calling a poison control center or doctor, or going for treatment.

If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after five minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

If on Skin or Clothing: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

NOTE. Do not use with a water maintenance cartridge or straw.

DIRECTIONS FOR USE. It is a violation of federal law to use this product in a manner inconsistent with its labeling. Always use Solution 1 and 2 together.

Applying ICX Renew to a Dental Unit's Waterlines

1. Turn off the delivery system master toggle; then remove and empty the water bottle.
2. Remove air/water syringe tips, handpieces, detachable motors and couplers, and ultrasonic instruments from their tubing.
3. Add one bottle (4 ounces) of ICX Renew Solution 1 and one bottle (4 ounces) of ICX Renew Solution 2 into the empty water bottle.
4. Cover the opening of the water bottle and carefully swirl the solution, coating all surfaces of the water bottle.
5. Reattach the water bottle to the unit and turn on the master toggle.
6. Run ICX Renew through the system until the blue solution flows out of the air/water syringe(s), all of the handpiece tubing and the cuspidor's cupfill waterline. Leave remaining solution in the water bottle.
7. Turn off the master toggle and allow the solution to remain in the unit for 12 to 96 hours.

NOTE Place a cloth or paper towel underneath the handpiece tubing to catch potential drips.

8. After 12 to 96 hours, turn on the master toggle and flush remaining ICX Renew solution through all air/water syringe(s), all handpiece tubing, and the cupfill waterline.
9. Turn off the master toggle, remove the water bottle, discard any remaining solution, and thoroughly rinse the bottle with water.
10. Refill the bottle with water, reattach it to the unit; then turn on the master toggle.
11. Completely flush all air/water syringe(s) and all handpiece tubing into a sink or cuspidor, and run the cupfill until the water is clear and the bottle is empty.
12. Once treatment with ICX Renew is completed, continue with your daily waterline maintenance protocol.

IMPORTANT. For **new equipment** apply one treatment before initial use. For **existing equipment**, apply three treatments when your water quality test results exceed your water quality action level. You may use the dental unit between treatments.

STORAGE AND DISPOSAL. Solutions 1 and 2:

Do not contaminate water, food, or feed by storage or disposal.

Product Storage. Keep the product in the closed original container when not in use. Store in a cool, dry area away from potential sources of heat, open flames, sunlight or other chemicals. Store between 5 °C (41 °F) and 30 °C (86 °F).

Product Disposal. Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

Container Handling. Nonrefillable container. Do not reuse or refill this container. Offer container for recycling if available or puncture and dispose of in a sanitary landfill, or by incineration.

ACTIVE INGREDIENTS SOLUTION 1:

Hydrogen peroxide:.....	2.1%
Other ingredients:.....	97.9%
<hr/>	
Total:	100.0%

SOLUTION 1:

EPA Registration No: 79662-2

Regulatory Information and Warranty: For required regulatory Information and the A-dec warranty, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at www.a-dec.com.

A-dec, A-dec 360, the A-dec logo, and ICX Renew are trademarks or registered trademarks of A-dec, Inc. in the U.S. and other countries.

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A-dec Headquarters

2601 Crestview Drive
Newberg, Oregon 97132
United States
Tel: 1.800.547.1883 within USA/CAN
Tel: +1.503.538.7478 outside USA/CAN
www.a-dec.com

A-dec Australia

Unit 8
5-9 Ricketty Street
Mascot, NSW 2020
Australia
Tel: 1.800.225.010 within AUS
Tel: +61.(0)2.8332.4000 outside AUS


A-dec China

A-dec (Hangzhou) Dental Equipment Co., Ltd.
Building 5, No.528 Shunfeng Road
Tangqi Town, Yuhang District
Hangzhou, Zhejiang, China 311100
Tel: 400.600.5434 within CN
Tel: +86.571.89026088 outside CN

A-dec United Kingdom

Austin House
11 Liberty Way
Nuneaton, Warwickshire CV11 6RZ
England
Tel: 0800.233285 within UK
Tel: +44.(0).24.7635.0901 outside UK

Donawa Lifescience
Consulting Srl
Piazza Albania, 10
00153 Rome, Italy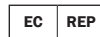

86.0741.00 Rev D
Date of Issue 2023-09-14
Copyright 2023 A-dec, Inc.
All rights reserved